Genius

Sistema antidecubito a bassa pressione continua a 3 zone differenziate *Low continuous pressure antidecubitus system with 3 differentiated zones*

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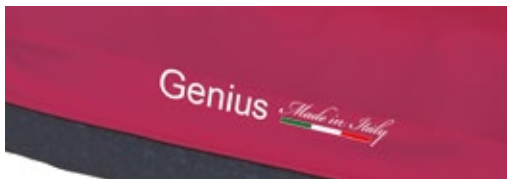
Genius is a continuous low-pressure anti-decubitus system with 3 differentiated sectors: head, trunk and legs. This allows to obtain an optimal contact pressure always lower than the capillary occlusion pressure in each body segment. Automatic pressure control.

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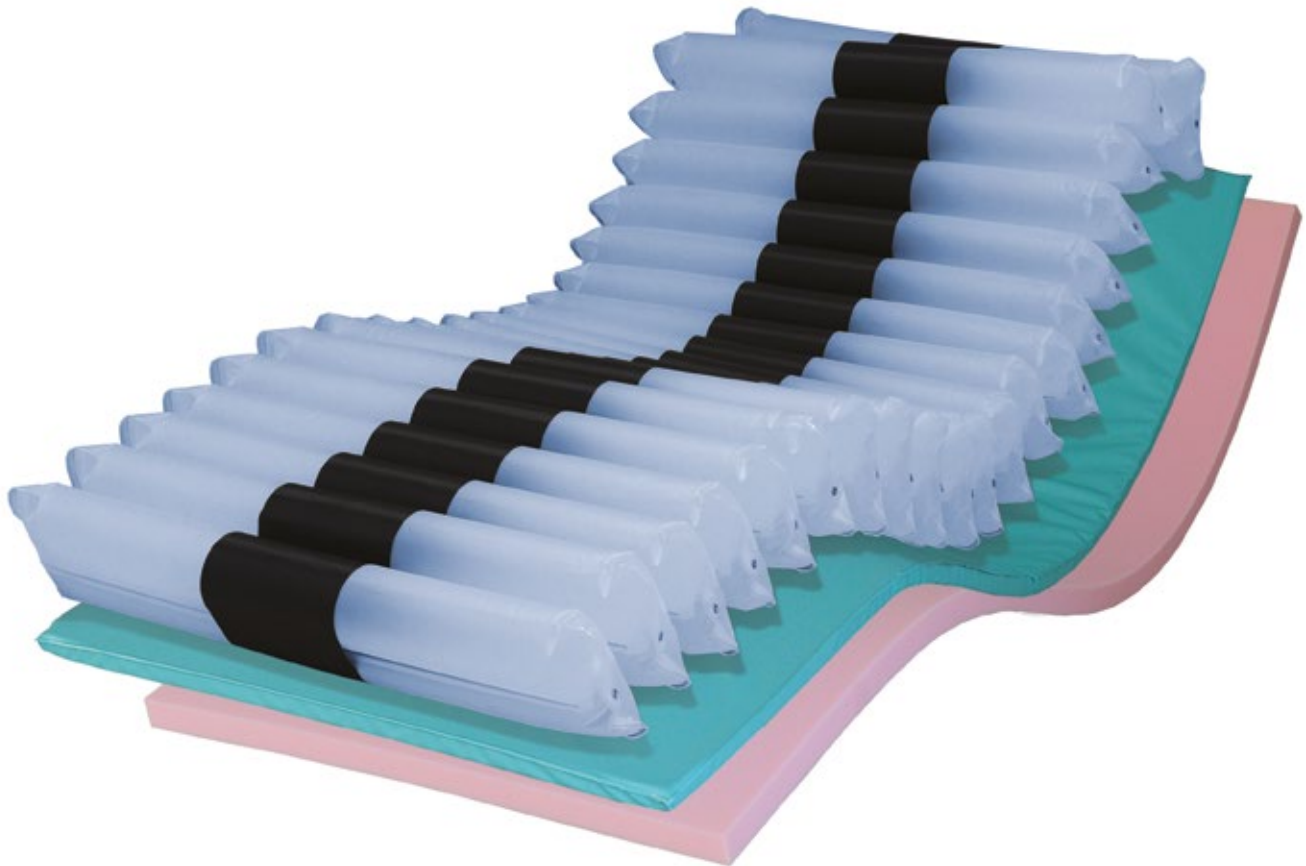
GENIUS is made of polyurethane foam base positioned under the 20 inflatable PU elements with an "8-shape" section divided in three zones:

- **3 elements** for the head zone
- **9 elements** for the body zone
- **8 singularly** excludable elements

The modular structure of the system allows to release each of the individual inflatable elements to obtain the complete discharge of the contact pressures in the desired area. Radiotransparent surface.

Breathable surface. CPR Cadrio Pulmonary Resuscitation function located at the head of the patient

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Technical features

- *Electronic and automatic regulation of inflation pressures in three independent zones*
- *Maximum inflation function*
- *Automatic keyboard lock*
- *Visual and acoustic alarm*
- *Low-noise operation*
- *Rechargeable battery to ensure 2 hours of operation in the event of a blackout*

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Sensors

Pressure sensors placed under the inflatable elements, which regulate the inflation of the three sectors on the basis of the weight and posture of the patient.

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Component materials are certified by the manufacturer as fire resistant. Genius systems are homologated as class 1IM

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The production is completely Italian. Our products are manufactured with high tech machinery, in order to quickly satisfy the customers' needs at high quality standard.

Management System
ISO 9001:2015
EN ISO 13485:2016

www.tuv.com
ID 9105025193

EUROAUSILI S.r.l.

Stabilimento e Show Room:
Via R. Cuttica, 43/45
20025 Legnano (MI) - Italy

tel. +39.0331.542850/0331.440931
fax +39.0331.547929

Stabilimento:
Via Genova, 21
20020 Villa Cortese (MI) - Italy

web: www.euroausili.it
email: info@euroausili.it

rev.0/2022

www.euroausili.it